


INSTRUCTIONS FOR JOEY CLAMP OPENING TOOL

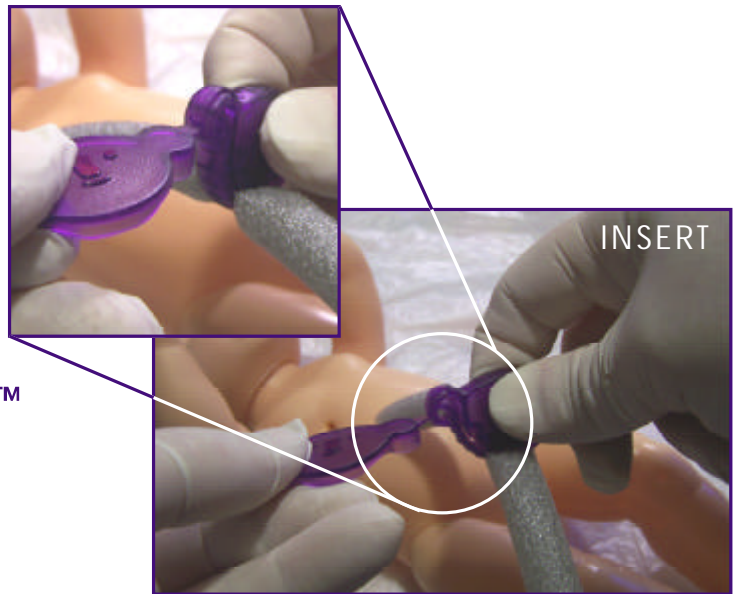

① To remove Joey Clamp, insert tip of the Joey Clamp Opening Tool (opener) into the slot located directly under the latch of the Joey Clamp. The latch is located on the clamp's right ear.

② Push the opener all the way into the slot releasing the latch. DO NOT PRY THE TOOL UP & DOWN. Its wedge shape is designed to open the latch with no rocking motion needed.

③ If repositioning clamp, to minimize risk of bleeding make sure that either (A) a separate clamping device is properly employed to clamp the cord proximal to the Joey Clamp being removed, or (B) the physician-prescribed time (e.g. 18-24 hrs.) has passed to ensure adequate closure of cord.

④ To re-close Joey Clamp, using two hands, place clamp around umbilical cord and, using two hands, squeeze until clasp has fully latched.

