

- ✓ CLICK
- ✓ CLAMP
- ✓ POP



Joey™

SPRAY GUARD

- ① Place the Joey Umbilical Cord Clamp and Cutter Safety System (Joey) on sterile field.
- ② Grip the device using the right hand, thumb at position Number 1, middle finger should be on the other side opposite the thumb.
- ③ Hold Joey with the colored clamp toward the infant and the opening of the device to the left. **ALIGN CORD** in the central **V-SHAPED** groove of the Joey. The colored infant clamp should be at least 1 centimeter away from the infant's intact skin. *Be certain that there is no tension on the cord.*
- ④ Using thumb and middle finger squeeze the two wings of the Joey together until the Number 1 latch is fully latched. A **"Click"** sound will be heard. (This action has cut the cord.)
- ⑤ Move the thumb forward on the device to the grip over the colored clamp indicated by the Number 2. Using thumb and index finger squeeze the two wings together until the Number 2 latch is fully latched. Again a **"Click"** sound will be heard. (This action has closed the infant **Clamp**.)
- ⑥ Visually verify that the cutter and clamp are fully latched. Using thumb, manually push out latched colored infant clamp from the latched cutting unit. **Pop**



I
N
S
T
R
U
C
T
I
O
N
S